

Ambulatory pump

TECHNOLOGY FOR HEALTH



CRONO *SC*

CRONO SC ambulatory pump has been designed for the subcutaneous infusion of drugs for the palliative care of terminal patients and for the subcutaneous infusion of Treprostinil.

PUMP DESCRIPTION

CRONO SC ambulatory pump has been designed for the subcutaneous infusion of drugs for the palliative care of terminal patients and for the subcutaneous infusion of Treprostinil.

The pump is indicated for subcutaneous use and allows for 4 administration methods:

1. Continuous
2. Bolus on request
3. Clinical bolus (managed by the doctor)
4. Combined (continuous + bolus on request + clinical bolus).

The pump is equipped with a settings lock to prevent incorrect or unauthorised administration. The liquid crystal display shows practical information to the doctor and patient about the settings, operation and diagnostics of the pump.

CRONO SC

TECHNICAL FEATURES

PUMP DIMENSIONS	76 x 49 x 29 mm (3.00 x 1.94 x 1.15 in).
PUMP DIMENSIONS (WITH PROTECTIVE WINGS)	138 x 49 x 29 mm (5.43 x 1.94 x 1.15 in).
WEIGHT	115 g (including battery).
WEIGHT (WITH PROTECTIVE WINGS)	127 g (4.46 oz.), including battery.
BATTERY	3 Volt Lithium battery, mod. 123 A (battery life approximately 100 infusions).
SINGLE-USE RESERVOIRS	Special 10 and 20 ml reservoirs with "Luer-Lock" universal safety fitting.
ADMINISTRATION DOSES	Selectable, from 1 ml to 20 ml in 1 ml increments.
PRIMING FUNCTION	Only available at the start of the infusion. The maximum volume that can be delivered is 1500 µl in 5 µl increments.
FLOW RATES SELECTABLE	From 5 to 2000 µl/h in 1 µl/h increments: It is possible to disable this function by setting 0 µl/h.
BOLUS DOSE	Selectable, from 10 µl to 2000 µl in 10 µl increments. It is possible to disable this function by setting 0 µl.
CLINICAL BOLUS	Selectable, from 10 µl to 2000 µl in 10 µl increments. It is possible to disable this function by setting 0 µl.
MINIMUM INTERVAL BETWEEN BOLUS DOSES	Selectable, from 5 min to 24 hours in the following modes: - from 5 minutes to 1 hour in 5 minute increments; - from 1 hour to 24 hours in 15 minute increments. It is possible to disable this function by setting no,Lt.
BOLUS DOSES IN ONE HOUR	From 0 to 12 bolus doses. The function may be disabled by setting no,Ln.
FLOW RATE PRECISION	+/- 2 %.
MAXIMUM OCCLUSION PRESSURE	2.5 bar +/- 1.5 (PL1); 4 bar +/-2 (PL2); 5.5 bar +/- 2 (PL3).
SETTINGS MEMORY	The settings are automatically entered in a flash memory which is retained even if the device is left without a battery.
DISPLAY	Liquid crystal display (dimensions 1.1 x 2.8 cm).
MOTOR	Coreless DC motor. The microcontroller controls the rotation speed using an infrared encoder.
SAFETY CIRCUITS	They check that the device is functioning correctly, intervening in the event of any fault with acoustic signals and messages on the display.
ANTI FREE-FLOW SYSTEM	To avoid a non controlled flow inside the delivery set due to the action of the force of gravity.
SUPPLIED STANDARD EQUIPMENT	Ambulatory pump, infusion pump carry-case, elastic belt, collar strap, fabric case, 2 batteries, battery cover opening tool, 2 user manuals (doctor and patient).
GUARANTEE	Two years for manufacturing defects.