


Crono S-PID 50 V3

Short Programming Guide - Timed Mode

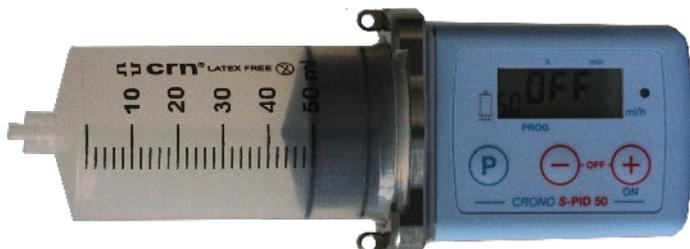

For models from
Serial Number:

PL6693.15

Caution: Users must familiarise themselves with the full user instructions. These notes provide only a reminder of main functions.

Pump activation

Your pump is supplied with battery installed and ready for operation.

The default delivery setting for the S-PID 50 is timed mode. This allows programming in time of infusion. This is the normal mode for UK use.

It is possible to change the mode of operation to program and show delivery in ml/hr but this should only be changed under care team instructions.

A separate instruction sheet is available on request for operation in flow rate mode

Programming the pump.

Before programming the pump press the **[P]** and **[-]** buttons together to ensure that all previous operations have been cancelled and the pump is able to receive new program instructions.

With the pump displaying **[OFF]** pressing the **[P]** button will allow you to step through a menu to set the following parameters in turn.

Infusion Time	Adjust as required
<i>End of infusion alarm</i>	<i>On/Off</i>
Syringe partial volume	1ml steps

Setting procedure

Press the **[P]** button to select Infusion time setting

*Use the **[+]** or **[-]** button to change the displayed value*

Press the **[P]** button to confirm value and then select alarm setting.

*Use the **[+]** to set alarm on or **[-]** button to select alarm off*

Press the **[P]** button to confirm value and select the partial volume setting.

*Press **[-]** button to reduce the volume or **[+]** button to increase.*

Press the **[P]** button to confirm value and the pump will take a short time to position the plunger at the programmed position.

Starting infusion and changing infusion time

Start the infusion by pressing the **[+]/ON** button.

*You are given the option to prime the infusion set **[Pr]**. Press and hold the **[P]** button to deliver the required prime up to a maximum of 1.5mL.*

Press the **[+]** button again to start the infusion at any time.

It is also possible to change the infusion time while the pump is delivering.

*To change the infusion time press the **[-]** button until the screen symbols flash, and then press the **[+]** button to increase, or the **[-]** button to reduce the infusion time.*

After 5 secs this will be locked into the pump memory.

Suspending Infusion

If you wish to pause your infusion at any time press the **[+]** and **[-]** buttons together, the word **[Stop]** will be displayed on the screen and an intermittent 'beep' will be heard. You can silence this sound by pressing the **[-]** button.

To resume infusion just press the **[+]** button.

You may terminate the infusion by disconnecting your infusion set and pressing the **[P]** and **[-]** buttons together.

End of infusion

If you have not terminated the infusion using the above procedure, at the end of the infusion the plunger will automatically return to its programmed start position..