

INSERTING THE INFUSION SET

1. Assemble the equipment you require.
2. Wash your hands.
3. Choose an insertion site as recommended by your healthcare provider. The choice of site depends on treatment and patient specific factors. Check that the insertion site is free of skin irritation such as redness, nodules, etc.
4. Clean the area of insertion with a disinfectant as directed by your healthcare provider. Be sure the area is dry before inserting the neria™ soft 90 infusion set.
5. Open the blister pack.
6. Remove the paper on the tubing.
7. Connect to pump/syringe by turning the luer-lock connector clockwise.
8. Grip the insertion handle and prime the tubing according to the pump manufacturer's instructions. Ensure there are no air bubbles in the tubing.
9. Pull gently to remove the backing paper.
10. Gently twist and pull to remove the needle guard.
11. Pinch the skin and insert the neria™ soft 90 infusion set at a 90 degree angle (straight in).
12. Press the adhesive onto the skin.
13. Remove the introducer needle by gently pulling the insertion handle straight up.
14. Secure the introducer needle by closing the insertion handle to cover the needle.
15. Dispose of the introducer needle in a sharps container.
16. Check the infusion site frequently to ensure that the soft cannula remains in place and that the site is not irritated. Replace the infusion set if necessary.


REMOVING THE INFUSION SET

- a. Wash your hands and stop the infusion.
- b. Disconnect the tubing from the pump/syringe by turning the luer-lock anti-clockwise.
- c. Remove the infusion set by carefully lifting the adhesive and working around the infusion set.
- d. Dispose of the infusion set in a sharps container.

Neria Soft 90

Short Instructions for Use

Note: These instructions are an extract from the manufacturer's full instructions for use. Users should familiarise themselves with the full contents of the manufacturer's document before using the device as it may contain additional important information.

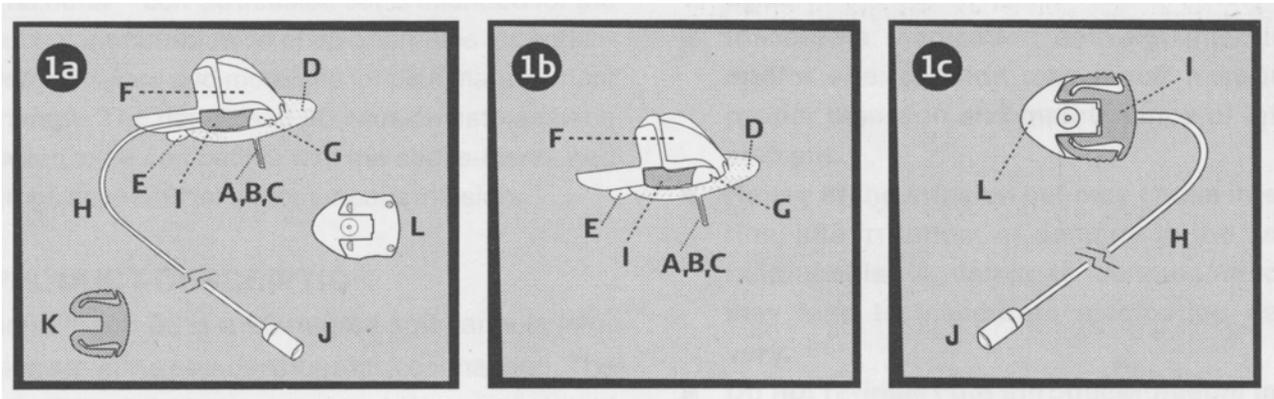

1 Contents

- | | |
|----------------------|--|
| A - Soft cannula | G - Cannula housing |
| B - Needle guard | H - Tubing |
| C - Introducer guard | I - Connector |
| D - Adhesive tape | J - Luer-lock connector |
| E - Backing paper | K - Disconnect cover (Cannula housing) |
| F - Insertion handle | L - Disconnect cover (tubing) |

INSERTING THE INFUSION SET

Please see full instructions over the page for insertion and removal procedures.

DISCONNECTING THE TUBING

- Wash your hands and stop the infusion.
- Place a finger in front of the cannula housing and gently squeeze the sides of the connector. Pull the connector straight out from the cannula housing.
- Place the disconnect cover (tubing) on the connector.
- Insert the disconnect cover (cannula housing) into the cannula housing until you hear a "click."

RECONNECTING THE TUBING OR USING A NEW TUBING

- Wash your hands.
- Remove the disconnect cover (cannula housing) by placing a finger in front of the cannula housing and gently squeezing the sides of the disconnect cover. Pull the disconnect cover straight out of the cannula housing.
- Remove the disconnect cover (tubing) on the tubing connector. Prime the tubing according to the pump manufacturer's instructions. Ensure there are no air bubbles in the tubing.
- Place the finger in front of the cannula housing and push the connector straight in until you hear a "click".