

# Crono S-PID 100

## Short Programming Guide - Flow Mode (mL/hr)



**Caution:** Users must familiarise themselves with the full user instructions. These notes provide only a reminder of main functions.

### Pump activation and mode selection

The default delivery setting for the S-PID 100 is timed mode. This allows programming in time of infusion. It is possible to change the mode of operation by removing and replacing the battery.

*The pump will then perform a short self test procedure during which a full screen display of all elements should appear.*

When the symbol [-t-] is displayed press and hold the [P] button until the display flashes. Press the [-] button until the display changes to [-F-]. Press [P] to confirm and the pump will switch to [OFF] mode.

### Locking/unlocking the keypad

By default keyboard lock is not activated on this device.

*With the pump displaying [OFF] press the (-) key for approximately 7 seconds. A number representing the number of completed infusions will appear on the screen.*

While still holding the [-] key depress the [+] key. The screen will display flashing [L.0] and keyboard is unlocked.

*Use the (+) or (-) button to select required value. (L1 is on, L0 is off)*



This symbol will display when the keyboard is locked.

## Programming the pump

With the keyboard unlocked, and the pump displaying [OFF] pressing the [P] button will allow you to step through a menu to set the following parameters in turn.

<i>Set Flow rate</i>	<i>0.2 mL/hr to 300 mL/hr</i>
<i>End of infusion alarm</i>	<i>On/Off</i>
Infusion Break Points	IP.1 to IP.5 (Normally set to IP.1)
<i>Syringe Partial Volume</i>	<i>1mL steps</i>

## Setting procedure

Press the [P] button to select flow rate setting. *Use the (+) or (-) button to set the desired rate*

Press the [P] button to confirm value and select the Alarm sound setting. *Press [-] button to turn the sound off or the [+] button to turn it on.*

Press the [P] button to confirm value and select the Infusion Break setting. *Press [-] button to reduce the number or the [+] button to increase.*

Press the [P] button to confirm value and select the partial volume setting. *Press [-] button to reduce the volume or the [+] button to increase.*

Press the [P] button to confirm value and the pump will take a short time to position the plunger at the programmed position.

## Starting infusion and setting infusion time

Start the infusion by pressing the [+ / ON] button.

*You are given the option to prime the infusion set [Pr]. Press and hold the [P] button to deliver the required prime up to a maximum of 1.5mL.*

Press the [+] button again to start the infusion at any time.

## Resetting plunger position

At the end of an infusion the plunger will automatically return to its programmed start position.

*To return this earlier, switch the pump into [StoP] mode by pressing the [+] and [-] keys together. Then press the [P] and [-] buttons together until [END] appears on the screen.*

The syringe rubber plunger will not be withdrawn with this procedure.