

Cane CRONO-30 V2 Program Guide

NOTE: These are short form instructions only. Users must familiarise themselves with the full Instructions as supplied with the pump, prior to use in a clinical situation.



Initialising pump

The pump is delivered with the following basic settings:

Alarm Sounds:	Active (On)
Syringe Partial Volume:	30mL

It is possible to use the pump programming features to change these parameters to suit individual requirements using the [P] button functions.

- Press and hold the [P] button and the alarm function will indicate [on] or [Off] depending on any previous setting. Press the [-] key to switch the alarm from [on] to [Off] or the [+] key to switch the other way.
- Press the [P] button to set this parameter and move on to the partial volume settings.
- If you require to deliver less than the full syringe volume use the [-] key to reduce the volume to the desired level. Once the value is set wait for 10 seconds and the pump will move the plunger forward to the correct setting. The pump will display the [OFF] screen.

Turning pump on

With the [OFF] screen displayed, press the (+) Button to turn the pump on. The display will show [Pr] to allow you to prime the pump. Press and hold the [P] key to deliver the desired prime up to a maximum of 1.5mL. Release the [P] key and press the [+] key at any time to start the main infusion.

Programming the pump

With the Keyboard unlocked, and the pump displaying [OFF] pressing the [P] button will allow you to step through a menu to set the following parameters in turn.

End of infusion alarm	On/Off
Syringe partial volume	1ml steps

Setting procedure

Press the [P] button to select alarm setting.

Use the (+) to set alarm on or (-) button to set alarm off

Press the [P] button to confirm value and select the partial volume setting.

Press [-] button to reduce the volume or [+] button to increase.

Press the [P] button to confirm value and the pump will take a short time to position the plunger at the programmed position.

Starting infusion and setting infusion time

Start the infusion by pressing the [+ / ON] button.

You are given the option to prime the infusion set [Pr]. Press and hold the [P] button to deliver the required prime up to a maximum of 1.5mL.

Press the [+] button again to start the infusion at any time.

To change the infusion time press the [-] button until the screen symbols flash, and then press the [+] button to increase, or the [-] button to reduce the infusion time.

After 5 secs this will be locked into the pump memory.

Resetting plunger position

At the end of an infusion the plunger will automatically return to its programmed start position

To return the plunger earlier, switch the pump into [StoP] mode by pressing the [+] and [-] keys together. Then press the [P] and [-] buttons together until [END] appears on the screen.

The syringe rubber plunger will not be withdrawn with this procedure.